MANUAL DE INSTRUCCIONES

Andador DEHESA

Manual de instrucciones

ES

Instruction manual

EN

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FR

Manuale D'Istruzioni

IT

Anweisungen

DE

Instruções

PT

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NL

Bruksanvisning

SWE

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PL

Brugsanvisning

DK

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NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

1. STEPS FOR THE USE OF THE ROLLATOR

1. Activate the rollator brakes by pressing the brake handles.
2. Place both feet together in the middle of the rollator frame.
3. Deactivate the brakes by lifting the brake handlebars.
4. Grasp both handlebars of the rollator.

2. TIPS AND PRECAUTIONS

- When standing up from a seated position, push yourself up from the seat instead of pulling on the handlebars of the rollator.
- Apply the brakes before sitting down and make sure that the brake is applied before getting up to use the rollator again.
- Take short steps to turn and keep your hips parallel to the front bar of the rollator.
- Avoid turning your body to the opposite side of the rollator.
- Make sure all wheels are on the ground before walking.
- Do not bend towards the rollator.
- Do not use on conventional or escalators.
- Do not place heavy objects in the basket, doing so may cause the rollator to tip over.
- Make sure that all parts are tight and that all four wheels are in contact with the ground before use.
- Maximum weight: 136kg.

3. INDICATIONS

It is designed to help the elderly, the disabled and people with walking problems to walk.

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FOSHAN SUNCARE MEDICAL PRODUCTOS CO., LTD.
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.