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## IMPORTANT

Keep the instructions for future reference. Read these instructions before assembling and using this product. Adult assembly is required.

Thank you for choosing the DIANA bed barrier! In this instruction manual you will find all the information you need to assemble and use this product correctly.

This rail has been developed with safety and comfort in mind, providing reliable support to facilitate the process of getting out of bed safely and effortlessly.

Before you start, we recommend that you read this manual carefully to ensure safe and optimal use of the product.

## NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

## PURPOSE OF THE PRODUCT

1. **Provide firm and stable support:** Facilitate the movement of getting into, out of or sitting on the bed with greater security and confidence.
2. **Reduce the risk of falls:** Provide a point of support to minimise accidents when moving in bed or trying to stand up.
3. **Increasing independence:** Helping older people to perform basic movements more autonomously, reducing the need for constant assistance.
4. **Providing comfort and safety:** Increasing the peace of mind of both the user and their caregivers by providing a safer environment.
5. **Encourage ergonomic use:** Designed to fit the physical needs of users, improving posture and reducing strain when getting up.

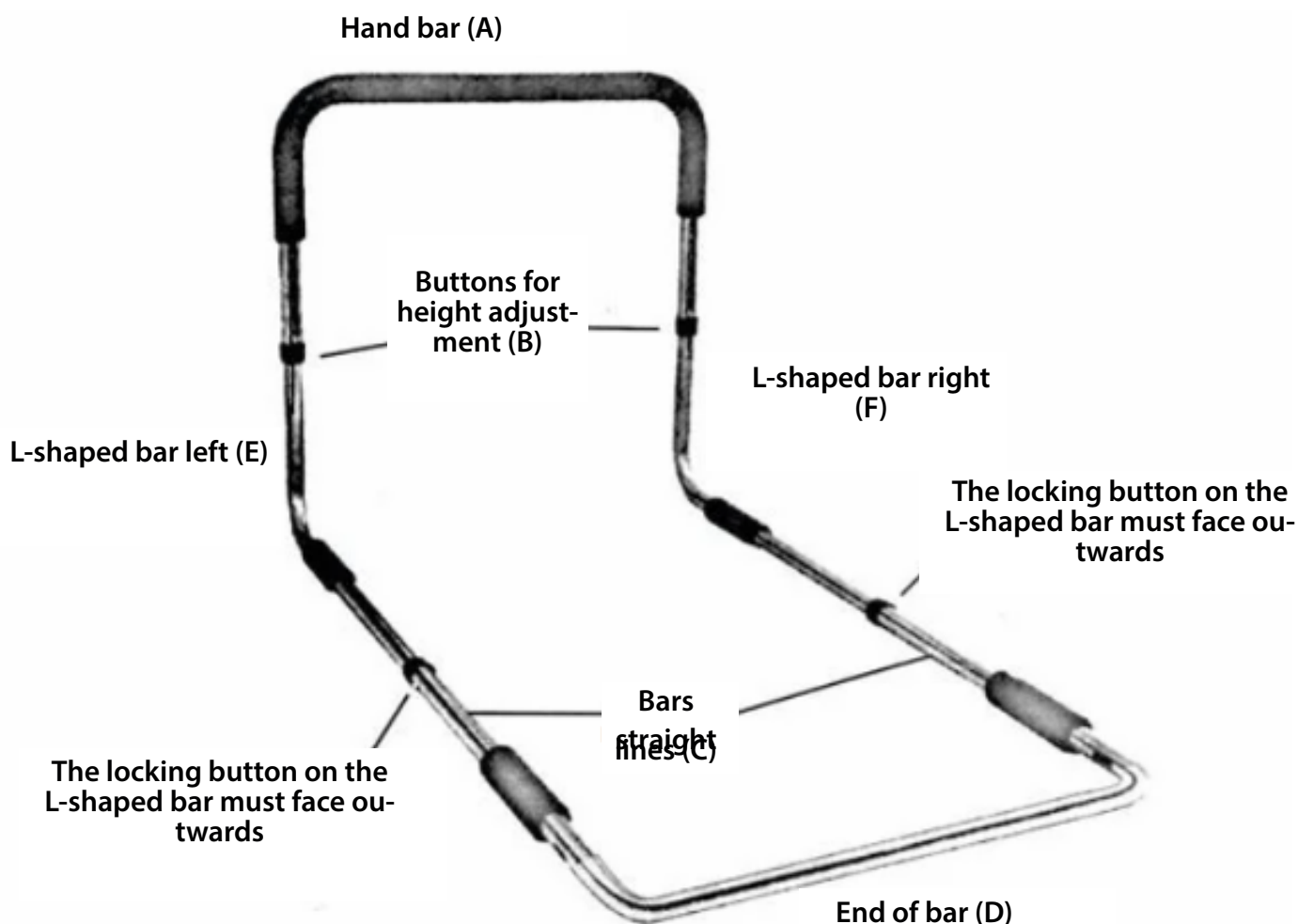
## 1. WARNINGS AND PRECAUTIONS

1. **WARNING!** This rail is not intended to be used as a barrier to prevent the user from falling out of bed.
2. **WARNING!** Check that the user does not exceed the specified maximum weight.
3. **WARNING!** Ensure that the handrail is installed correctly according to the instructions before use. Incorrect installation may result in accidents or injury.
4. **WARNING!** This product is not intended for use by children and should not be used as a play item.

## 2. PRODUCT FEATURES

|                                 |                     |
|---------------------------------|---------------------|
| <b>PRODUCT NAME</b>             | Bed railing         |
| <b>MODEL</b>                    | DIANA               |
| <b>MODEL REFERENCE</b>          | MD-00007/31         |
| <b>MAXIMUM SUPPORTED WEIGHT</b> | 286,60 lb           |
| <b>NET PRODUCT WEIGHT</b>       | 4,62 lb             |
| <b>PRODUCT DIMENSIONS</b>       | 37,00" * 20,07"     |
| <b>MATERIAL</b>                 | Chrome-plated steel |
| <b>BELT MATERIAL</b>            | Nylon               |

### 3. PARTS AND COMPONENTS

### 4. INDICATIONS

**This product is intended for:**

- Older people at risk of falls.
- Patients with limited mobility or reduced strength.
- Patients recovering from surgery or with injuries that affect their balance.

### 5. CONTRAINDICATIONS

**This product is not intended for:**

- People with advanced dementia, severe confusion or impulsivity who may attempt to climb the railing. This could increase the risk of falls from greater heights and serious injury.
- People who are completely dependent on assistance to turn or stand up could become trapped or injured if unsupervised.

- Patients with arm, shoulder or wrist injuries, which could be aggravated by using the support as a fulcrum.
- Users with general paralysis.
- Children and/or babies.

## 6. ASSEMBLY INSTRUCTIONS

**Step 1:** Remove all parts from the packaging and arrange them as shown in figure 1.

**Step 2:** Place the L-shaped bars in such a way that the height adjustment knob is visible towards the outside as shown in figure 1 (E) and (F).

**Step 3:** Insert each straight bar (C) into each L-shaped bar until it clicks into position (Align pin with locking button position).

**Step 4:** Place the handrail over the small end of the L-shaped bars and press down until the locking button engages. Adjust for height as required.

**Step 5:** Connect the bar end (D) to the straight bars (C). Make sure that they fit together correctly

**Step 6:** Once the product is assembled, place between the mattress and the bed base and adjust the height of the rail as required.

**REGULAR HEIGHT**

To adjust the height, press both buttons at the same time until the desired height is reached.

**STRAP INSTRUCTIONS:**

1. Unbuckle both parts of the strap by pressing on the inside of the buckle and pulling them apart. Pass the strap through the outer loop upwards, then slide the strap down through the centre of the pin into the buckle. Repeat the same action for the other side of the buckle.
2. Lift the side of the mattress closest to the end rail (D).
3. To install the strap, place the strap under the bed base and then attach both parts of the strap to the top of the bed base.
4. Then adjust the straps to fix the rail to the bed base.

## 7. INSTRUCTIONS FOR USE

- Make sure that the rail fits snugly against the mattress in the L-curve with the handle upwards.
- The handrail is designed to help the user get into, out of and onto the bed correctly.

## 8. MAINTENANCE AND CLEANING

- Clean the product regularly with a dry or slightly damp cloth.

## 9. PROBLEMS AND SOLUTIONS

**Problem:** The handrail is loose or unstable

**Solution:** Check that the railing is assembled according to the instructions.

**Problem:** The rail does not fit well to the bed

**Solution:** Adjust the position of the railing according to the installation options offered.

**Problem:** Visible wear or damage to parts.

**Solution:** Periodically inspect the railing parts for cracks, deformation or corrosion.

**Problem:** Difficulty in assembling the product.

**Solution:** Carefully review the step-by-step assembly instructions

## 10. CONDITIONS OF GUARANTEE

This product is covered by a 36 month warranty against manufacturing defects.

Thank you very much for having trusted Mobiclinic for this DIANA bed rail! We value your trust in our products and are committed to always offering you the best quality.

If you have any questions or need additional assistance, please do not hesitate to contact us through our customer service at **clientes@mobiclinic.com**. We appreciate your comments and suggestions for further improvement.

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XIAMEN NEAQ IMP.& EXP. CO., LTD
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.